


UV-C Mobile Surface Disinfection Unit

UV-MAX

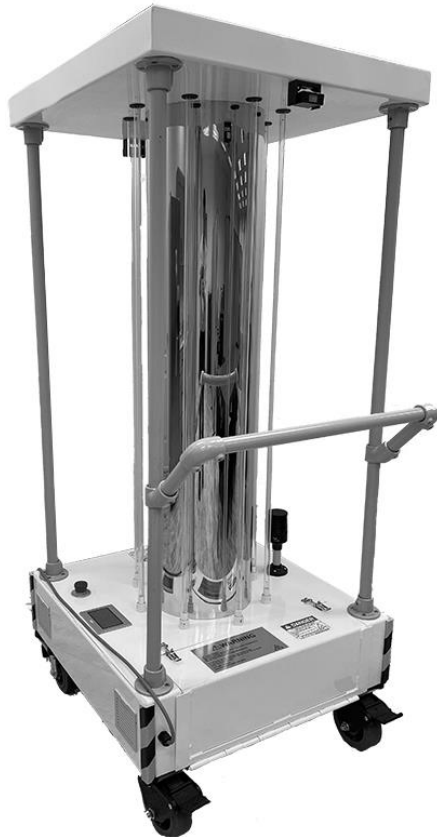

LAMP REPLACEMENT GUIDE

Models:

- UV-MAX "MAX"
- UV-MAX "LO PRO"
- UV-MAX "ECO"
- UV-MAX "ULTRA"


Step 1: Position Device

- Position device and lock wheels to prevent rolling
- Ensure device is disconnected from power source
- Wear proper PPE when handling lamps (gloves and safety glasses)
- Open control panel drawer and slide out panel tray




Step 2: Removing Lamp

- Identify ballast on panel that corresponds with same number lamp
- Remove white and black wires connected to that ballast for same corresponding lamp (slide small screwdriver into lock slots below wires to free from ballast)
- Snip zip-ties in wire bundle taking care not to cut any wires
- Lift up on lamp being removed removing it from holding slot and pull down out of upper holding grommet
- Gently pull wires attached to removed lamp through wire bundle
- Properly dispose of the damaged / non-functioning lamp (same process as florescent lamps)

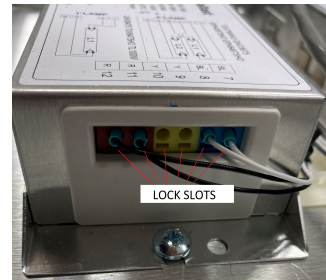

Step 3: Installing Lamp

- Trim 1/4" of wire casing from end of new lamp wires
- Feed wires for lamp into holding slot for that lamp while slowly pulling wire set through from below
- Gently insert top of lamp into top panel grommet and seat bottom of lamp into base holding slot
- Run wires for lamp along wire bundle and insert into corresponding ballast (slide small screwdriver into lock slots to open locks for accepting wires; remove it to lock)
- Ensure wires are securely attached to ballast before closing panel drawer
- If desired, fasten new zip ties around wire bundles

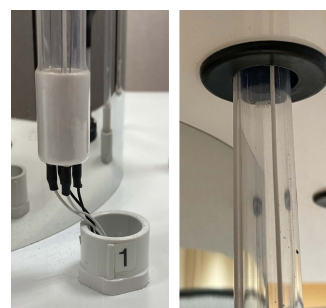



Step 4: Test Lamp

- Position Device with PIR sensor toward entryway
 - Connect device to power supply
 - Initiate treatment cycle (see User Guide)
 - Leave treatment space immediately
 - Close door and place warning signage
 - Observe lamp operation through window from adjoining room or place video recording device in treatment space
- check lamp operation status
- Avoid direct exposure to UV light
 - Do not look at UV light without approved safety glasses

Additional Information:

Please contact manufacturer for:

- Troubleshooting
- FAQ's
- Diagnostics

ENERGENICS.COM



1470 DON STREET
NAPLES, FL 34104
SALES@MAX-ASSURE.COM
1.239.643.2778